



# Dyna CAD/CAM - **exocad** Library Guide

Modelcreator for Model Printing exocad® Imetric®

Dutch Technology



— your comfort is our goal!

## Dyna Helix Implant Modelcreator Library Installation guide for exocad® Imetric® for Model Printing

### Installation:


- Take file Dyna\_Helix\_ExoCad\_Modelcreator\_V21.3.zip
- Extract directory files to the modelcreator directory of exocad:  
..... DentalCADApp\library\modelcreator\implants

### Detailed file location:


To find the exact location on your system you can open the correct DentalCADApp file location by clicking the right mouse button on the Exo DentalDB icon on your desktop.




- Then click on the highlighted location,

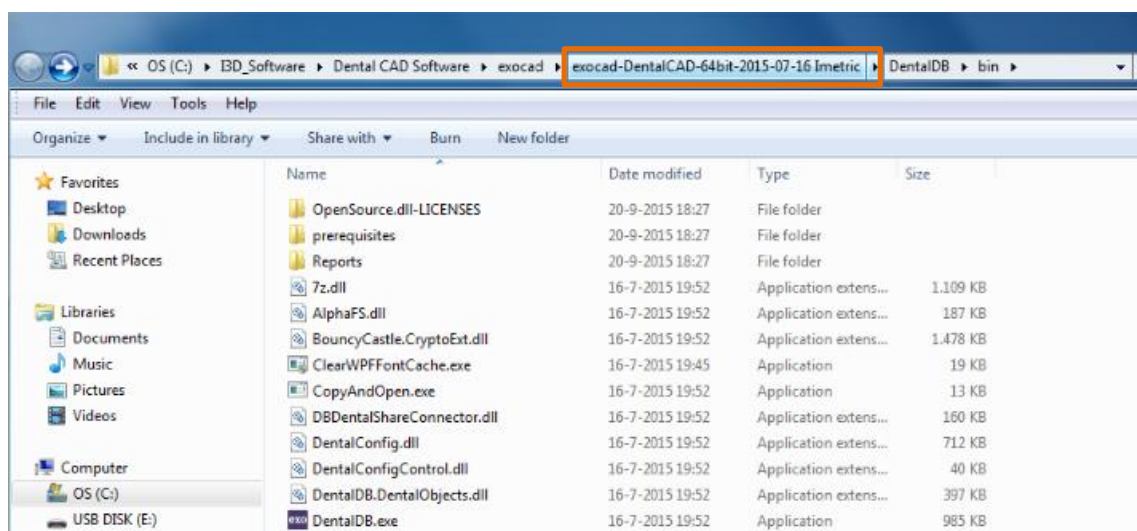

- Next, open the DentalCADApp location,

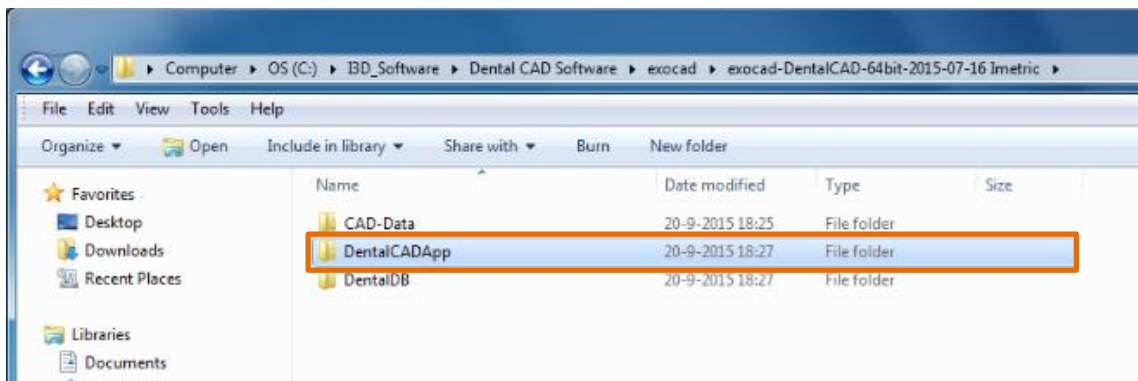

- Then open the library location,

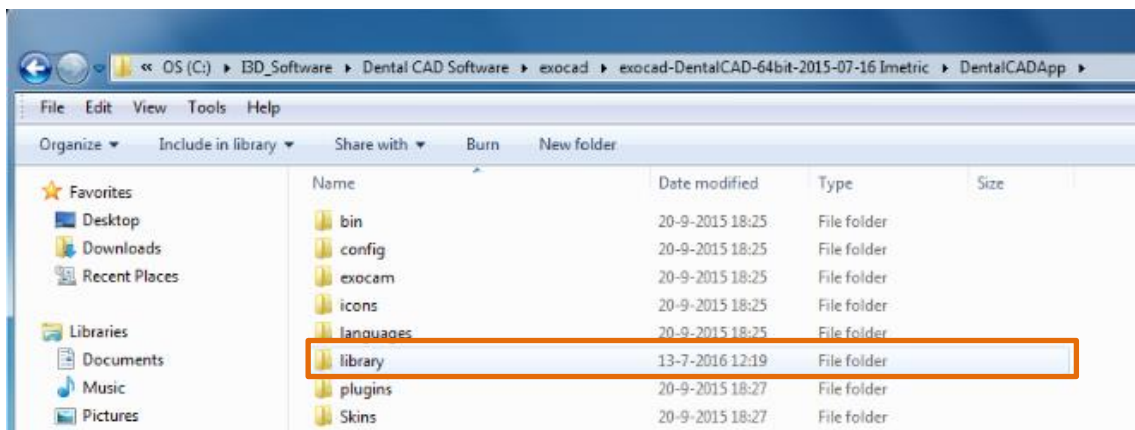

- Finally open the modelcreator location.

