



Dyna CAD/CAM - **exocad** Library Guide

Accurate, individual manufactured constructions

Dutch Technology



— your comfort is our goal!

Dyna Helix Implant Library Installation guide for exocad® Imetric®

Installation:


- Take file Dyna_Helix_Library_Exocad_V1_19.zip
- Extract directory files to: C:\I3D_Software\Dental CAD Software\exocad\exocad-DentalCAD-.....Imetric\DentalCADApp\library\implant

Detailed file location:


To find the exact location on your system you can open the correct DentalCADApp file location by clicking the right mouse button on the Exo DentalDB icon on your desktop.




- Then click on the highlighted location,

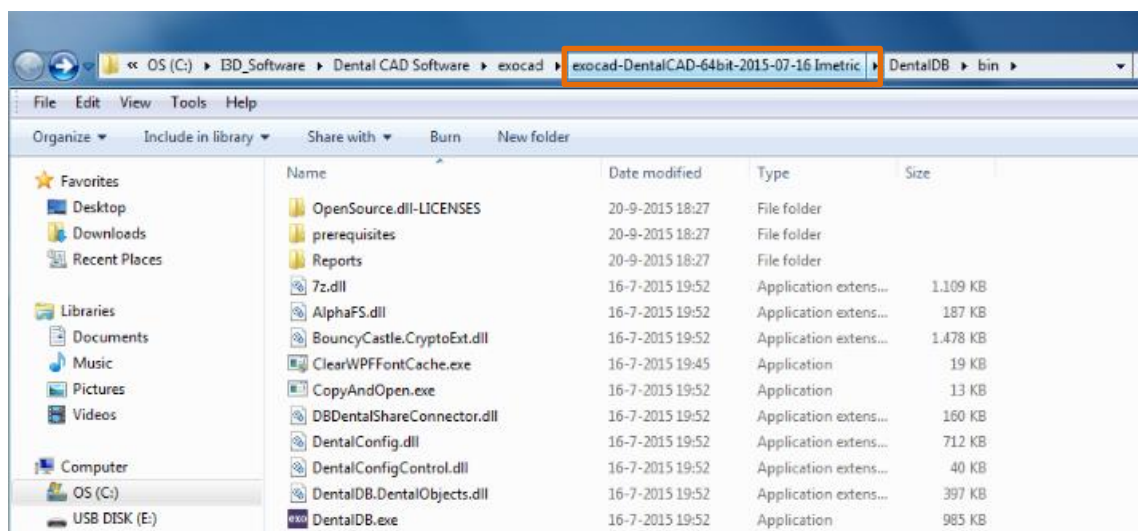


- Next, open the DentalCADApp location,

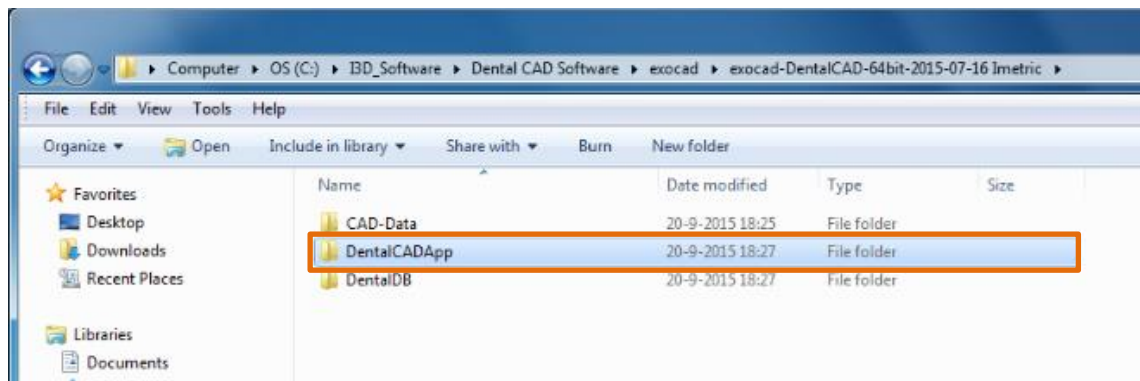

- Then open the library location,

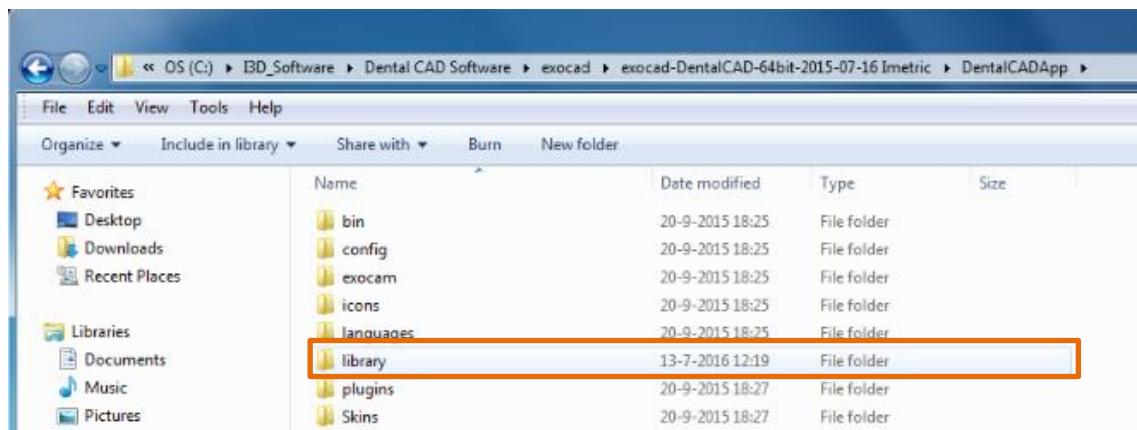


- Finally open the implant location.

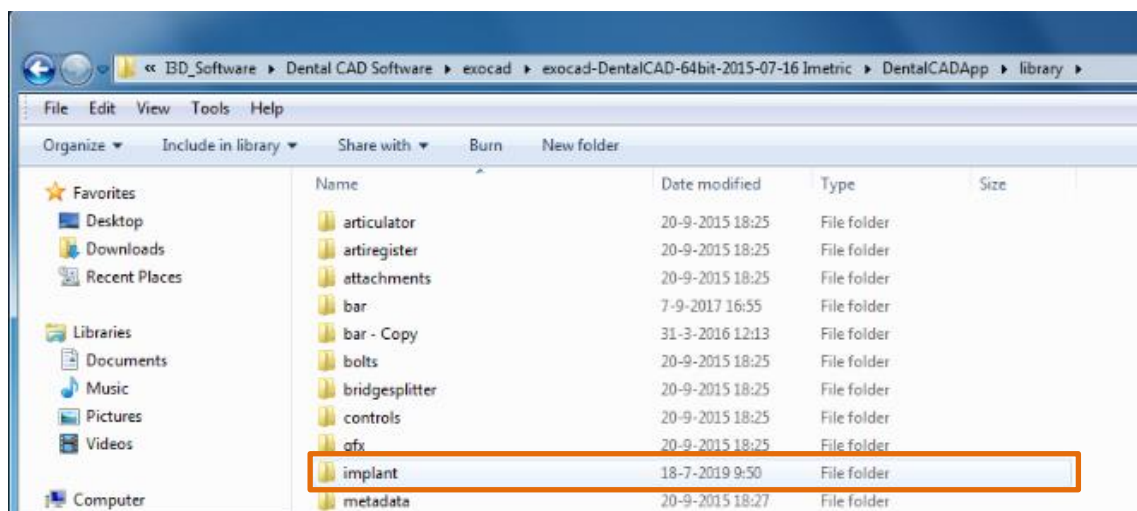


Use of the Library:

The library is not issued by Exocad and because of this the program will warn you when using this library with the following notes:

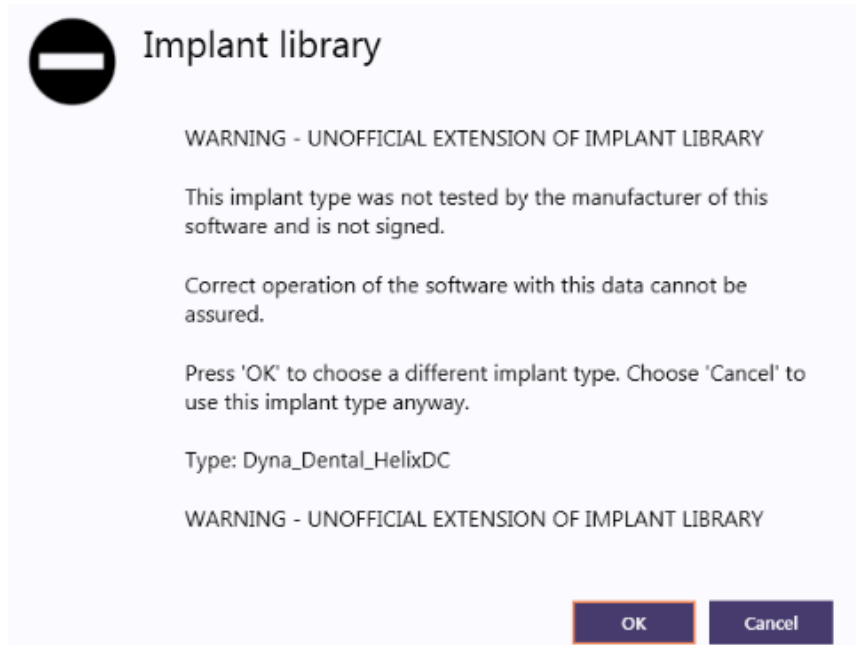

To continue choose Cancel. After choosing cancel the following warning will pop up:

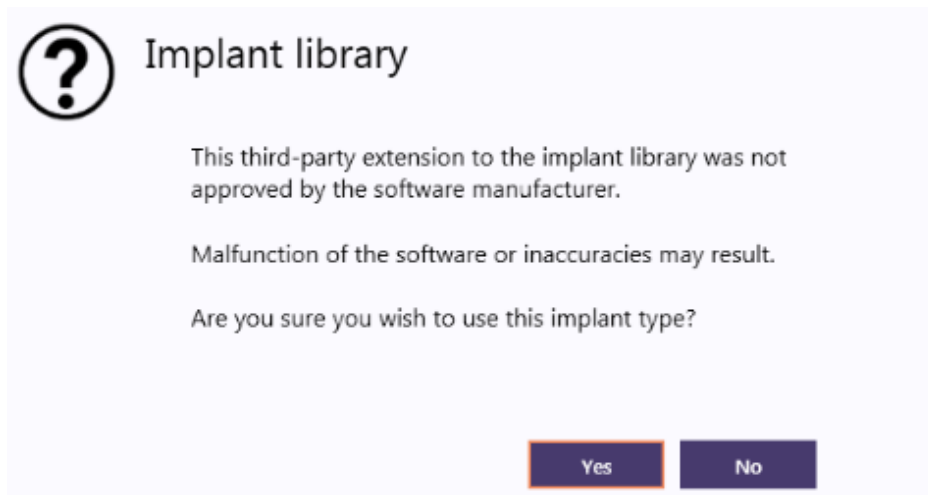

Just choose Yes to continue.

The rest functions according normal use.